



Instruction Manual
Compressors

CLINIC 1.25

CLINIC DRY 1.25

CLINIC 3.25

CLINIC DRY 3.25

CE EAC

EN



INDEX

INTRODUCTION	4
STATEMENTS AND TESTING	5
Statement of delivery of the instruction manual	5
Testing and commissioning certificate	6
EC DECLARATION OF CONFORMITY	7
STATEMENT AND CE MARK	8
Statement and CE mark	8
GENERALI	9
Importance of the manual	9
Consultation notes	10
State "machine off"	10
Authorized operators	11
Abbreviations	13
All rights reserved	13
Warranty	14
TECHNICAL DESCRIPTION	15
Name of models	15
Names of parts	16
Dimensions of the machine	18
Technical data	19
Intended use of the machine	21
Source of energy supply	21
Products used for the operation	21
Phases of operation	22
Reasonably foreseeable misuse	22
TRANSPORTATION AND HANDLING	23
Transportation of the machine	23
Packaging	23
Handling of packaging with a forklift or transpallet	24
Unpacking	24
Manual handling of the machine	25
Positioning of the machine	26
INSTALLATION	27
General warnings	27
Pneumatic connection	27
Electric connection	28
SAFETY	29
Safety devices	29
Personal protective equipment (PPD)	31
Residual risks	31
Safety signs	32
USE AND OPERATION	33
Places occupied by authorized operators	33
Pressure switch	34
Prior to ignition	36
Turning on the machine	36



Resettable thermal 37
 Turning off the machine 38
 Emergency stop..... 38
 Activation after an emergency stop 39
 Intervention of the safety valve..... 39
TROUBLESHOOTING..... 40
MAINTENANCE 41
 Every week 42
 Every month 43
 Dryer filter ESC. F1..... 44
 Dryer filter ESC. F3..... 45
 Extraordinary maintenance..... 46
DECOMMISSIONING 46
 Storage 46
 Demolition 46
SPARE PARTS 47
MAINTENANCE SCHEDULE..... 48÷51
SINGLE TABLE FOR PACKAGING DISPOSAL..... 52

INTRODUCTION

Dear Customer, first of all, we wish to thank you for the confidence you gave us by purchasing your new “COMPRESSOR”.

We are sure that your expectations will be met, given the level of technology achieved by our products with a continued commitment on a daily basis, which encourages us to grow, to know how to deal with the constant changes in technology expertise, manufacturing and sales. We feel sure we can meet your future requirements, therefore, feel free to contact us. We will be happy to offer you all our experience and knowledge to answer any question you may have.

INFORMATION TO THE CUSTOMER

DOCUMENT	PAGE	TO DO
STATEMENT OF DELIVERY OF THE INSTRUCTION MANUAL	5	The authorised operators must complete and sign to confirm the delivery and reading of the instruction manual.
TESTING AND COMMISSIONING CERTIFICATE	6	Give to the test technician of the Manufacturer or to the Authorized Dealer a copy of page 6 completed in all its parts.
PRODUCT REGISTRATION FOR WARRANTY VALIDATION	14	See paragraph “WARRANTY” - Section 2) Page 14.



STATEMENTS AND TESTING

STATEMENT OF DELIVERY OF THE INSTRUCTION MANUAL

The employer, to the extent required by applicable law on safety and health of workers at work, hereby declares it has delivered to the authorized operators this manual for the correct information and training on use and maintenance of the machine.

AUTHORIZED OPERATORS	DATE	SIGNATURE FOR RECEIPT
Mr. _____	_____	_____

Before filling in the table, photocopy it for future reference.



TESTING AND COMMISSIONING CERTIFICATE	DATE _____
Designation: compressor	Name: _____
Model: _____	Street: _____
Serial No: _____	City: _____
Manufacture year: _____	State: _____
<p>This is to certify that the machine in question was tested by: Test technician of the manufacturer or authorized dealer:</p> <p>Mr. _____</p> <p>At the presence of the entrusted operator:</p> <p>Mr. _____</p> <p>At the presence of the entrusted operator and Safety Representative:</p> <p>Mr. _____</p>	
<input type="checkbox"/> Check for proper installation of the machine.	
<input type="checkbox"/> Verification of the correct setting of all the working parameters.	
<input type="checkbox"/> Verification of correct operation and intervention of all the safety devices.	
<input type="checkbox"/> Commissioning and performance of a work cycle.	
<input type="checkbox"/> Possible optimization of the working parameters.	
<input type="checkbox"/> Machine stop.	
<input type="checkbox"/> Delivery of the instruction manual.	
<p>SIGNATURE Test Technician of the Manufacturer or Authorized Dealer</p>	<p>Safety Representative</p>

GENTILIN S.r.l.

Via delle Tezze 20/22 - 36070 Trissino (VI) Italy
P.Iva/VAT 01262520248

[IT] **Dichiara che la seguente macchina, compressore volumetrico a pistoni senza lubrificazione collegato coassialmente a motore elettrico asincrono è conforme alle disposizioni delle seguenti direttive**

2006/42/CE Direttiva del Parlamento Europeo e del Consiglio del 17 maggio 2006, relativa alle macchine e che modifica la direttiva 95/16/CE (rifusione).

Direttiva **2014/29/UE** del Parlamento Europeo e del Consiglio del 26 febbraio 2014.

Direttiva **2014/30/UE** del Parlamento Europeo e del Consiglio del 26 febbraio 2014, che sostituisce la 2004/108/CE Direttiva del Parlamento Europeo e del Consiglio del 15 dicembre 2004, concernente il ravvicinamento delle legislazioni degli Stati Membri, relative alla compatibilità elettromagnetica e che abroga da Direttiva 89/336/CEE 97/23/CE (Art. 3.3).

Direttiva **2014/35/UE** del Parlamento Europeo e del Consiglio del 26 febbraio 2014, che sostituisce la 2006/95/CE del Parlamento Europeo e del Consiglio del 12 dicembre 2006.

Direttiva **2000/14/CE** del Parlamento Europeo e del Consiglio, del 8 Maggio 2000 e successive integrazioni

È conforme alle norme armonizzate EN 1012, EN 60204-1, EN 60335-1.

Persona autorizzata a costituire il Fascicolo Tecnico: **Gentilin Giuseppe**, in qualità di Responsabile Tecnico

[DE] **erklärt, dass die folgenden Maschinen, Volumenkolbenkompressor ohne Verschmierung koaxial mit einem asynchronen Elektromotor verbunden mit den Vorschriften der folgenden Richtlinien übereinstimmen:**

Richtlinie **2006/42/CE** des europäischen Parlaments und des Rates vom 17. Mai 2006 über Maschinen und zur Änderung der Richtlinie 95/16/EG (Neufassung).

Richtlinie **2014/29/UE** des europäischen Parlaments und des Rates vom 26. Februar 2014.

Richtlinie **2014/30/UE** des europäischen Parlaments und des Rates vom 26. Februar 2014, die die Richtlinie 2004/108/EG des europäischen Parlaments und des Rates vom 15. Dezember 2004 zur Angleichung der Rechtsvorschriften der Mitgliedstaaten über die elektromagnetische Verträglichkeit und zur Aufhebung der Richtlinie 89/336/EWG 97/23/EG (Art. 3.3) ersetzt.

Richtlinie **2014/35/UE** des europäischen Parlaments und des Rates vom 26. Februar 2014, die die Richtlinie 2006/95/EG des Europäischen Parlaments und des Rates vom 12. Dezember 2006 ersetzt.

Richtlinie **2000/14/CE** des europäischen Parlaments und des Rates vom 8. Mai 2000 und nachfolgende Ergänzungen.

Entspricht harmonisierten normen EN 1012, EN 60204-1, EN 60335-1.

Zur Erstellung der technischen Unterlagen bevollmächtigte Person: **Gentilin Giuseppe**, als technischer Leiter.

[FR] **Déclarer que la machine suivante: compresseur d'air à pistons volumétriques sans huile raccordé coaxialement à un moteur électrique asynchrone, est conforme aux dispositions des directives suivantes**

2006/42/CE Directive du Parlement européen et du Conseil du 17 mai 2006 sur les machines et modifiant la directive 95/16/CE (refonte).

2014/29/UE Directive du Parlement européen et du Conseil du 26 février 2014

2014/30/UE Directive du Parlement européen et du Conseil du 26 février 2014, remplaçant la directive 2004/108/CE du Parlement européen et du Conseil du 15 décembre 2004, concernant le rapprochement des législations des États membres relatives à la compatibilité électromagnétique et abrogeant Directive 89/336/CEE.

2014/35/UE Directive du Parlement européen et du Conseil du 26 février 2014, remplaçant la directive 2006/95/CE du Parlement européen et du Conseil, du 12 décembre 2006.

2000/14/CE Directive du Parlement européen et du Conseil du 8 mai 2000 et compléments ultérieurs.

Conforme aux normes harmonisées EN 1012, EN 60204-1, EN 60335-1.

Personne habilitée à préparer le Dossier Technique: **Gentilin Giuseppe**, en qualité de Responsable Technique

Modello / Matricola
Model / Serial number
Modell- / Seriennummer
Modello / número de serie
Modèle / numéro de série
Model / numer serijnyj

[EN] **Declares that the following machine, volumetric piston air compressor oil-less coaxially connected to an asynchronous electric motor, complies with the provisions of the following directives**

2006/42/EC Directive of the European Parliament and the Council of 17 May 2006 on machinery and amending Directive 95/16/EC (recast).

2014/29/UE Directive of the European Parliament and of the Council of 26 February 2014

2014/30/UE Directive of the European Parliament and of the Council of 26 February 2014, replacing 2004/108/EC Directive of the European Parliament and of the Council of 15 December 2004, on the approximation of the laws of the Member States relating to electromagnetic compatibility and repealing Directive 89/336/EEC.

2014/35/UE Directive of the European Parliament and of the Council of 26 February 2014, replacing Directive 2006/95/EC of the European Parliament and Council, of December 12, 2006.

2000/14/CE Directive of the European Parliament and the Council of 8 May 2000 and subsequent additions.

Complies with harmonized standards EN 1012, EN 60204-1, EN 60335-1.

Person authorized to prepare the Technical Dossier: **Gentilin Giuseppe**, in quality of Technical Manager

[ES] **Declara que las siguientes máquinas, compresor de pistón volumétrico sin lubricación conectado coaxialmente a un motor eléctrico asincrónico son conformes a las disposiciones de las siguientes Directivas**

2006/42/CE Directiva del Parlamento Europeo y del Consejo del 17 de mayo de 2006 relativa a las máquinas y que modifica la Directiva 95/16/CE (refundida).

2014/29/UE Directiva del Parlamento Europeo y del Consejo del 26 de febrero de 2014

2014/30/UE Directiva del Parlamento Europeo y del Consejo del 26 de febrero de 2014, que sustituye la 2004/108/CE Directiva del Parlamento Europeo y del Consejo del 15 de diciembre de 2004 relativa a la aproximación de las legislaciones de los Estados miembros sobre compatibilidad electromagnética y que deroga la Directiva 89/336/CEE 97/23/CE (Art.3.3).

2014/35/UE Directiva del Parlamento Europeo y del Consejo del 26 de febrero de 2014, que sustituye la Directiva 2006/95/CE del Parlamento Europeo y del Consejo del 12 de diciembre de 2006.

2000/14/CE Directiva del Parlamento Europeo y del Consejo del 8 de mayo de 2000 y adiciones subsiguientes.

Cumple con las normas armonizadas EN 1012, EN 60204-1, EN 60335-1.

Persona autorizada a constituir el Fascículo Técnico: **Gentilin Giuseppe**, en calidad de responsable técnico.

[PL] **Oświadczam, że następująca maszyna: wolumetryczna tłokowa sprężarka powietrza, bezolejowa, współosiowo połączona z asynchronicznym silnikiem elektrycznym, jest zgodna z postanowieniami następujących dyrektyw**

2006/42/WE Dyrektywa Parlamentu Europejskiego i Rady z dnia 17 maja 2006 r. W sprawie maszyn zmieniająca dyrektywę 95/16/WE (wersja przekształcona).

2014/29/UE Dyrektywa Parlamentu Europejskiego i Rady z dnia 26 lutego 2014 r

2014/30/UE Dyrektywa Parlamentu Europejskiego i Rady z dnia 26 lutego 2014 r. Zastępująca dyrektywę 2004/108 / WE Parlamentu Europejskiego i Rady z dnia 15 grudnia 2004 r. W sprawie zbliżenia ustawodawstw państw członkowskich odnoszących się do kompatybilności elektromagnetycznej i uchylająca Dyrektywę 89/336 / EWG.

2014/35/UE Dyrektywa Parlamentu Europejskiego i Rady z dnia 26 lutego 2014 r. Zastępująca Dyrektywę 2006/95 / WE Parlamentu Europejskiego i Rady z dnia 12 grudnia 2006 r.

2000/14/WE Dyrektywa Parlamentu Europejskiego i Rady z dnia 8 maja 2000 r. Wraz z późniejszymi uzupełnieniami.

Zgodny z normami zharmonizowanymi EN 1012, EN 60204-1, EN 60335-1.

Osoba upoważniona do przygotowania dokumentacji technicznej: **Gentilin Giuseppe**, kierownik ds. Technicznych

Firma (Legale rappresentante)
Signature (Legal Representative)
Unterschrift (gesetzlicher Vertreter)
Firma (Representante legal)
Signature (representant légal)
Podpis (przedstawiciel prawny)





STATEMENT AND CE MARK

MARKING AND TECHNICAL DATA

The CE marking (**Fig. 1**) of the machine indicates compliance with the essential occupational health and safety rules set forth by the European Directives listed in the Declaration of CE Conformity.

The label is black and white printed on adhesive polymeric PVC and has the following dimensions: L = 70 mm - H = 55 mm.

It is applied externally on one side of the machine indicated in **Fig. 1** and states the following data, in a legible manner:

- LOGO, NAME AND ADDRESS OF MANUFACTURER
- CE MARKING
- DESIGNATION OF THE MACHINE (type / model)
- YEAR OF MANUFACTURE
- SERIAL NUMBER
- S1 VALUE
- MAX. WORKING PRESSURE (bar)
- VOLTAGE (V / Hz / A)
- SHAFT ROTATION SPEED (rpm)
- RATED POWER (kW)
- WEIGHT (kg)


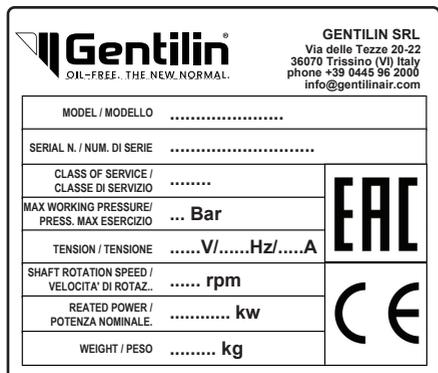


Fig. 1



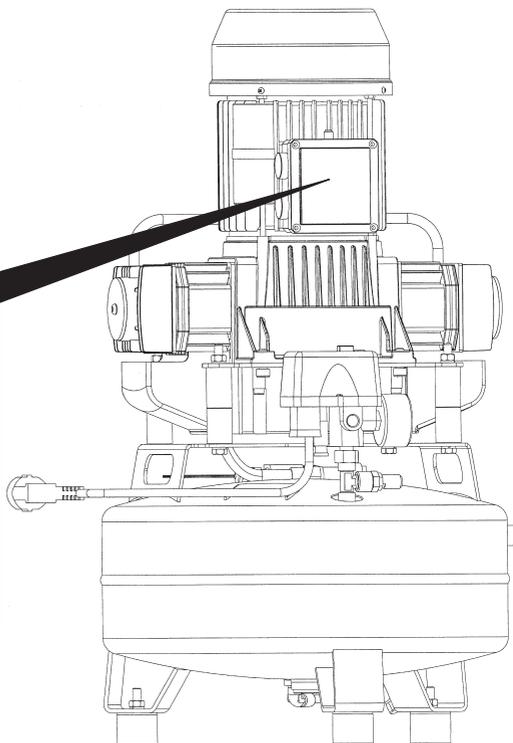

GENERAL

IMPORTANCE OF THE MANUAL



Before using the machine in question, the authorized operators are obliged to read and thoroughly understand this manual in all its parts.

This instruction manual has been prepared in accordance with the directions provided by the European Directives in order to ensure an easy and correct understanding of the topics covered by the authorized operators operating and maintaining the machine in question.

If, despite the attention being paid by the manufacturer while drafting this document, such operators should not fully understand this text while reading, they are requested to ask the manufacturer for the correct explanations and information in order to avoid incorrect personal interpretations that may jeopardize safety.

Before using the involved machine, the authorized operators are obliged to read and understand all parts of this manual and adhere strictly to the rules herein described in order to assure their own and others' safety, the best performance of the machine and ensure the maximum efficiency and durability of all its components.

This manual shall be available at any time for the authorized operators and shall be well protected and preserved, always close to the machine.



This manual should always be available for authorized operators and located in the vicinity of the machine well protected and preserved.

This manual must compulsorily be delivered together with the machine when it is transferred to another user.

The manufacturer disclaims all liability for injury to persons, animals and/or property caused by ignoring the rules and warnings described in this manual.

This manual reflects the state of the art of technique at the time of the machine marketing and cannot be considered improper just because, according to new experience, it may be updated later on.

In case of loss or damage of the manual request a copy to the manufacturer, specifying the identification data of the document: code, issuance and review.






CONSULTATION NOTES

SIMBOL	MEANING
	Signal of generic danger: It highlights risks to health and safety of operators and/or risk of damage or malfunction of the machine.
	Signal of generic obligation: It indicates a requirement (obligation to take action).
	Generic prohibition sign: It highlights the prohibition to perform an action.
	Danger Sign EX: It highlights the risk due to explosion.

STATE "MACHINE OFF"

-  Before performing any type of maintenance and/or setting on the machine, disconnect the power supply ensuring that the switch **(14)** is in position "OFF (0)", the plug of the power cable **(11)** is unplugged and placed near the machine and the tank has been drained from residual compressed air by opening the drain valve **(9)**.
- 

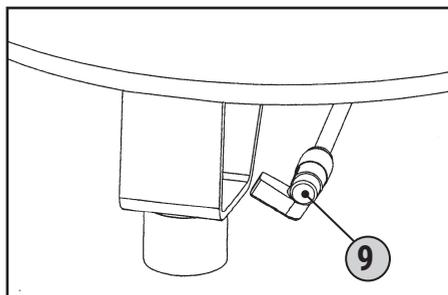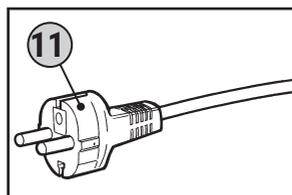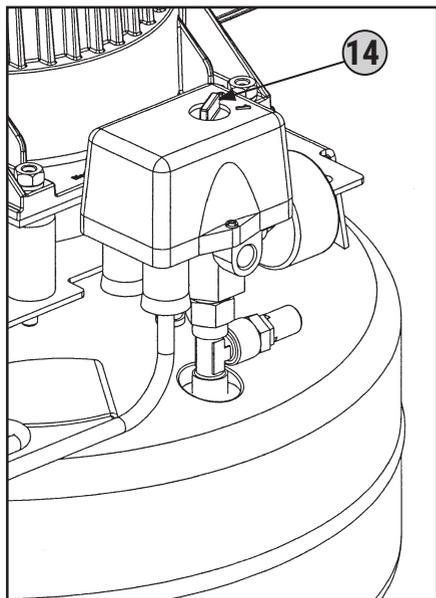



AUTHORIZED OPERATORS



THE OPERATORS ARE AUTHORIZED TO MAKE ON THE MACHINE ONLY SPECIFIC ACTIONS OF THEIR COMPETENCE.

THE AUTHORIZED OPERATORS, BEFORE PERFORMING ANY WORK ON THE MACHINE MUST BE SURE TO BE IN POSSESSION OF FULL PSYCHO-PHYSICAL CAPACITIES, SUCH AS TO ENSURE COMPLIANCE WITH THE SAFETY CONDITIONS.

This technical manual is intended exclusively for Authorized Operators to the use and maintenance of machines according to specific technical and professional skills required for the type of intervention. The symbols below are arranged at the beginning of a chapter and/or paragraph, to indicate which is the operator concerned with the subject matter.

OPERATOR IN CHARGE



It is a professionally trained operator, authorized to perform only the ignition, use, setup, tuning (with the mandatory protection enabled and the machine turned off) and the turning off of the machine and in total respect of the instructions in this manual, equipped with **personal protective equipment (PPD)** specified and occupying the places (see related paragraph).

OPERATOR IN CHARGE FOR HANDLING



It is a professionally trained operator, who has completed 18 years of age, in accordance with the law in the country of use, enabled to the conduct of forklifts, cranes or bridges wagons, to make safe transport, handling and unpacking the machine and/or parts of it, with the **personal protective equipment (PPD)** specified and occupying the places described (see related paragraph).

MECHANICAL/HYDRAULIC/PNEUMATIC MAINTENANCE TECHNICIAN



It is a qualified technician, authorized to perform actions only on the mechanical/hydraulic/pneumatic parts to make adjustments, and maintenance and/or repairs, even with the protections disabled (upon consent of the Supervisor) in strict compliance with the instructions in this manual or other document specifically provided exclusively by the manufacturer, equipped with **personal protective equipment (PPD)** specified and occupying the places described (see related paragraph).



ELECTRICAL MAINTENANCE TECHNICIAN



It is a qualified technician (an electrician in possession of technical and professional qualifications required by the regulations), authorized to perform only work on electrical equipment to make adjustments, and maintenance and/or repairs, even in the presence of voltage and with the protections disabled (upon consent of the Supervisor), **equipped with personal protection devices (PPD)** envisioned and occupying the positions described (see related paragraph).

SAFETY REPRESENTATIVE



It is a qualified engineer appointed by the customer in possession of technical and professional qualifications required by the regulations on safety and health of workers in the workplace.

MANUFACTURER'S TECHNICIAN



It is a qualified technician made available by the manufacturer and/or authorized dealer to perform the required technical assistance, routine work and extraordinary maintenance and/or operations not mentioned in this manual that require specific knowledge of the machine, equipped with **personal protective equipment (PPD)** specified (see related paragraph).



ABBREVIATIONS

Lists some abbreviations used in this book.

- cca. = About
- cap. = Chapter
- PPE = Personal protective equipment
- R = Right
- h = Hours
- EN = European Norm
- Es. = Example
- Fig. = Figure
- max. = Maximum
- min. = Minimum
- min = Minutes
- N. = Number
- page = Page
- par. = Paragraph
- Pos. = Position
- Rif. = Reference
- sec. = Seconds
- L = Left
- TAB. = Table
- see = SEE

ALL RIGHTS RESERVED

The reserved rights related to this instruction manual are in possession of the manufacturer. All information (text, drawings, diagrams, etc.) given here is confidential.

No part of this manual may be reproduced and distributed (in whole or in part) by any means of reproduction (photocopy, microfilm or otherwise) without the written consent of the Manufacturer.

All trademarks are property of their respective owners.



GUARANTEE

- 1) **GUARANTEE DECLARATION:** the Manufacturer gives an undertaking to the End-User to replace, repair or otherwise intervene in the machine, if it has conformity defects that compromise its correct use and operation, but only if such defects are attributable to the actual liability of the Manufacturer. The Manufacturer reserves the right to adopt the optimal solution to restore the conformity of the machine within a reasonable period of time. The guarantee does not cover parts of the Products subject to normal wear and tear and/or deterioration or defects resulting from inappropriate transport.
- 2) **END-USER:** the End-User is the person who buys the machine to use it directly. The End-User who buys the machine as a natural person for purposes that do not fall within the scope of commercial or professional activity is defined as a “consumer”. The End-User is defined as a “professional or company” when the purchase is made by a professional or by a company acting for purposes which fall within the scope of its commercial or professional activity.
- 3) **TERMINALS:** the Manufacturer is liable to the End-User when the lack of conformity occurs within the following terms from the date of purchase:
24 months if the purchase was made by a ‘consumer’ as described in point 2).
12 months if the purchase was made by a “professional or company” as described in point 2).
All End-Users have the right to extend the guarantee (Extended Guarantee) for up to three years from the date of purchase of the machine if they: (a) have registered their purchase on the website www.gentilnair.com within seven days of purchase and (b) have completed the registration form completely and accurately, filling in the required fields with exact, complete and true information and have kept the registration code to be mentioned in the event of problems. Replaced parts or products will remain under guarantee until the original product guarantee expires.
- 4) **VALIDITY:** the guarantee is valid if the End-User notifies the Manufacturer of the conformity defect within 2 months from the date on which he has discovered it, but within the final term indicated in point 3) above. The above report must be accompanied by a regular purchase document (tax receipt or invoice). In the case of an Extended Guarantee, the declaration must also indicate the registration code.
- 5) **FORFEIT:** the guarantee will be void if the purchaser uses the machine improperly and/or not in accordance with the provisions of this “Instruction Manual” provided by the Manufacturer, or if the lack of conformity was caused by inexperience on the part of the purchaser or by chance. The use of the products after the discovery of defects releases the Manufacturer from any guarantee commitment or liability.
- 6) **LIABILITY:** the Manufacturer is released from any liability resulting from any damage sustained by the purchaser, resulting from loss or diminution of production which derives from any lack of conformity.
- 7) **COSTS:** the End-User has the right to the restoration of the conformity of the machine under guarantee without any costs for labour or materials. The shipping methods must be agreed with the authorised dealer or with the Manufacturer. In the event that the non-conformity is only apparent or the complaint arrives after the deadline, the Manufacturer may charge the cost of both transport and any repair that is indicated by the authorised dealer or the Manufacturer.



TECHNICAL DESCRIPTION

NAME OF MODELS

The machine is offered in eight models. **Tab. 1** shows the model names and specifications that distinguish them.

MODEL	DESCRIPTION
CLINIC 1.25	<ul style="list-style-type: none">• Oil-free air compressor with 25 litres tank without dryer.
CLINIC DRY 1.25	<ul style="list-style-type: none">• Oil free air compressor with 25 litres tank with silica salt dryer.
CLINIC 3.25	<ul style="list-style-type: none">• Oil-free air compressor with 25 litres tank without dryer.
CLINIC DRY 3.25	<ul style="list-style-type: none">• Oil free air compressor with 25 litres tank with silica salt dryer.

Tab. 1




NAMES OF PARTS


In figures are represented the major components that make up the machine.

KEY

- 1) Support feet
- 2) Non-return valve
- 3) Single cylinder pumping unit
- 4) Electric motor
- 5) Restorable circuit breaker
- 6) Air intake filters
- 7) Dryer/filter
- 8) Air tank
- 9) Condensate drain valve
- 10) Tank pressure manometer
- 11) Electric power supply cable with plug
- 12) Safety valve (calibrated at 10.5 bar)
- 13) Pressure switch
- 14) "ON (I) - OFF (O) switch"
- 15) Blind fitting for accessories
- 16) Handle
- 17) Two-cylinder pumping unit

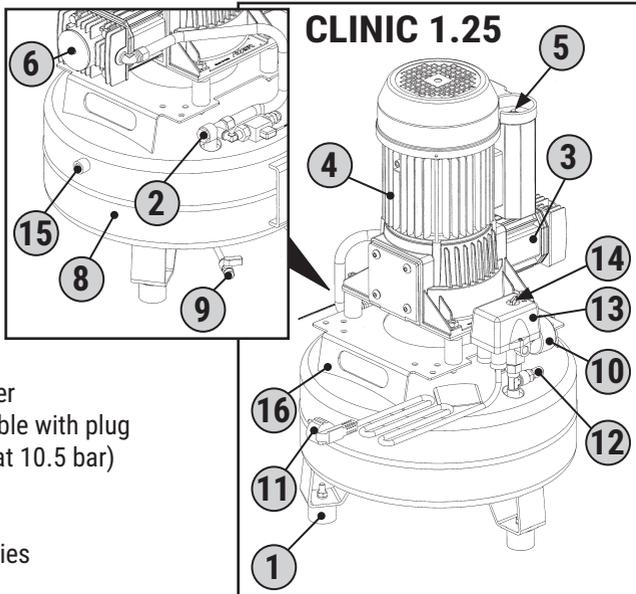


CLINIC DRY 1.25

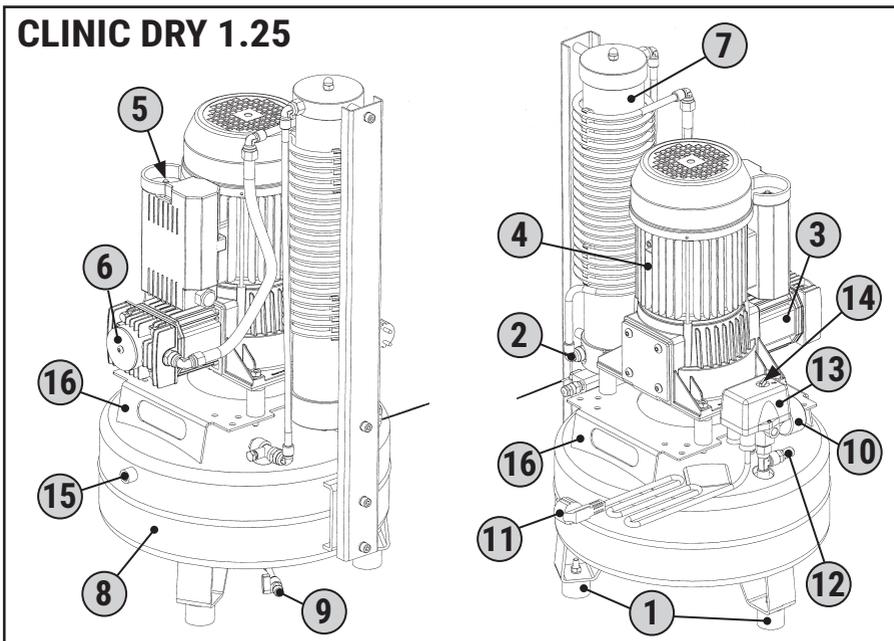




CLINIC 3.25

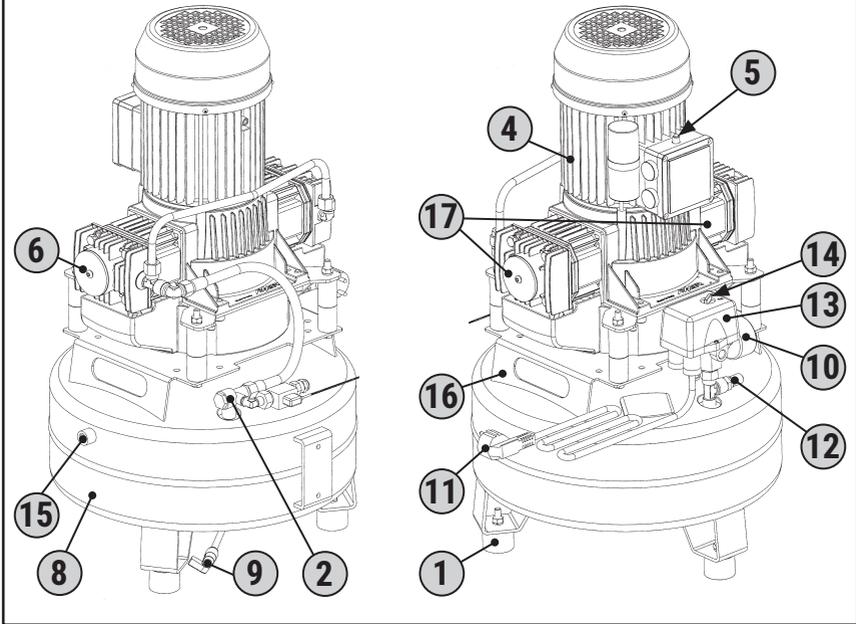

CLINIC DRY 3.25

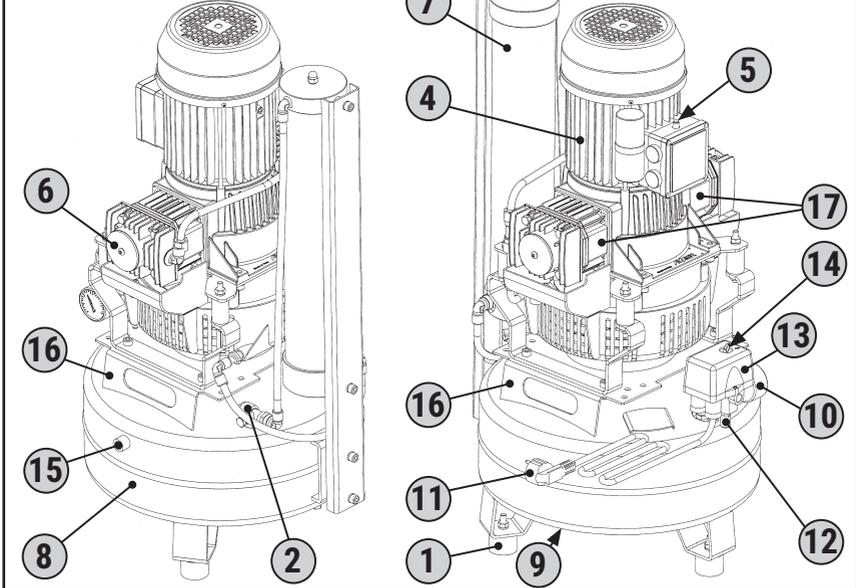





DIMENSIONS OF THE MACHINE

Tab.1 gives the clearance of the machine represented in Fig.1.


CLINIC 1.25

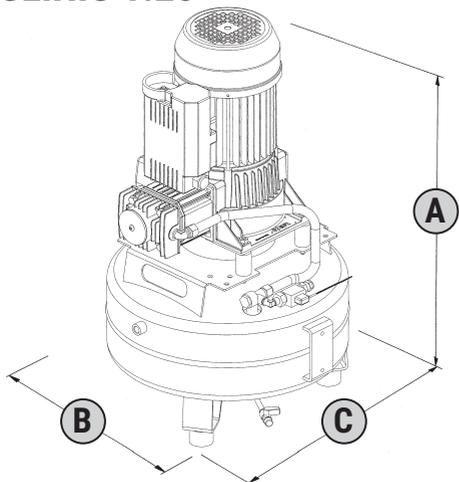

CLINIC DRY 1.25

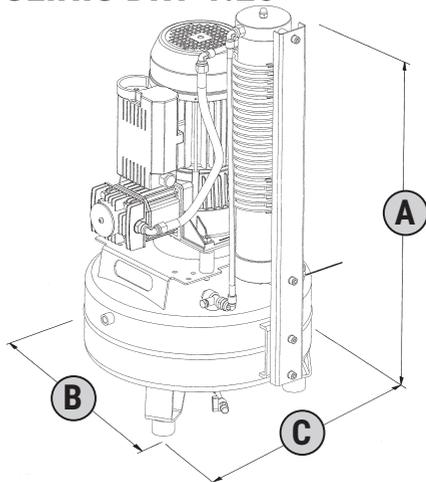

CLINIC 3.25

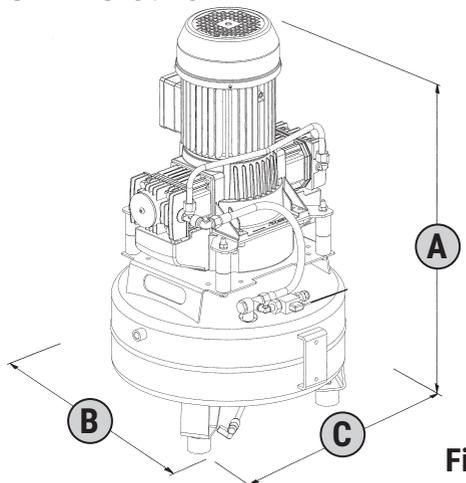

CLINIC DRY 3.25

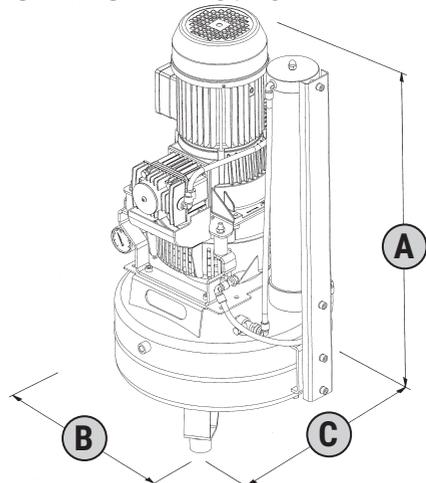


Fig. 1

MODELS	A	B	C
CLINIC 1.25	697 mm	420 mm	420 mm
CLINIC DRY 1.25	810 mm	466 mm	420 mm
CLINIC 3.25	741 mm	420 mm	415 mm
CLINIC DRY 3.25	825 mm	502 mm	415 mm

Tab. 1



TECHNICAL DATA

MACHINE DATA

Tab. 2 shows the technical data which characterize the machine.

CLINIC 1.25			
Power supply	230V/50 Hz	230V/60 Hz	110V/60 Hz
Electrical Absorption	6 amper	7,5 amper	14 amper
Units served	2	2	2
Air delivered (5 bar) (liters/min)	65	70	70
Intake air (liters/min)	130	150	150
Tank capacity (liters)	25	25	25
Motor power (Watt)	1000	1200	1200
Noise (dB(A))	64	64	64
Weight (Kg)	30,5	31,5	36
Ambient temperature (°C)	0-40	0-40	0-40
Service class	70%	70%	70%
Motor rotation direction	The compressor can be used in both directions of rotation		

CLINIC DRY 1.25			
Power supply	230V/50 Hz	230V/60 Hz	110/V60 Hz
Electrical Absorption	6 amper	7,5 amper	14 amper
Units served	2	2	2
Air delivered (5 bar) (liters/min)	65	70	70
Intake air (liters/min)	130	150	150
Tank capacity (liters)	25	25	25
Motor power (Watt)	1000	1200	1200
Noise (dB(A))	64	64	64
Weight (Kg)	38,5	39,5	44
Ambient temperature (°C)	0-40	0-40	0-40
Service class	50%	50%	50%
Motor rotation direction	The compressor can be used in both directions of rotation		

Tab. 2



CLINIC 3.25			
Power supply	230V/50 Hz	230V/60 Hz	110V/60 Hz
Electrical Absorption	8,1 amper	11 amper	22 amper
Units served	2	2	2
Air delivered (5 bar) (liters/min)	150	170	170
Intake air (liters/min)	270	320	320
Tank capacity (liters)	25	25	25
Motor power (Watt)	1800	2000	2000
Noise (dB(A))	67	67	67
Weight (Kg)	36	37	40
Ambient temperature (°C)	0-40	0-40	0-40
Service class	100%	100%	100%
Motor rotation direction	The compressor can be used in both directions of rotation		

CLINIC DRY 3.25			
Power supply	230V/50 Hz	230V/60 Hz	110V/60 Hz
Electrical Absorption	8,1 amper	11 amper	22 amper
Units served	2	2	2
Air delivered (5 bar) (liters/min)	150	170	170
Intake air (liters/min)	270	320	320
Tank capacity (liters)	25	25	25
Motor power (Watt)	1800	2000	2000
Noise (dB(A))	67	67	67
Weight (Kg)	43	44	47
Ambient temperature (°C)	0-40	0-40	0-40
Service class	50%	50%	50%
Motor rotation direction	The compressor can be used in both directions of rotation		



INTENDED USE OF THE MACHINE

The machine was designed and built for the use described in **Tab. 1**.

FIELD OF USE	Dentistry, laboratories or dental cabinets.
PLACE OF USE	Places indoors with sufficient lighting, ventilation, air environment having the characteristics referred to under par. "MACHINE DATA" and "PRODUCT USED FOR OPERATION" appropriate to the laws in force in the country of use for safety and health in the workplace. The machine must be placed on a plan that ensures stability in relation to the overall dimensions and weight, respecting the minimum position shown in par. "POSITIONING OF THE MACHINE". WARNING! YOU MUST KEEP THE MACHINE OUT OF REACH OF UNAUTHORIZED PERSONS.
INTENDED USE	Compressed air production to operate dental tools.
OPERATORS IN CHARGE FOR USE	An authorized operator with the technical and professional capacities meeting the requirements.

Tab. 1

SOURCE OF ENERGY SUPPLY

Tab. 2 describes the source of energy supply of the machine.

ENERGY SOURCE	POWER SUPPLY
ELECTRIC	Connection via plug on electrical switchboard. (See Technical Data)

Tab. 2

PRODUCTS USED FOR THE OPERATION

Tab. 3 shows the product used for the operation.

PRODUCTS USED FOR THE OPERATION
Ambient air "clean" (good quality) free of pollutants harmful to health
Intake air filters (paper)
Dryer /filters

Tab. 3



PHASES OF OPERATION

In **Tab. 4** are listed the steps of operation of the machine (please see the relative paragraphs).

PHASE	OPERATION
1	Turning on the machine.
2	Adjusting the outlet pressure.
3	If requested reset the resettable thermal.
4	Turning off the machine.

Tab. 4

REASONABLY FORESEEABLE MISUSE

The machine was designed and built for use provided, therefore it is absolutely prohibited any other type of application and use, in order to ensure at all times the safety of the Authorized operators and the efficiency of the machine itself.



RISK OF BURNS DUE TO ACCIDENTAL CONTACT WITH THE PUMPING GROUP AND ELECTRIC MOTOR AND PUMP UNIT. WARNING THERE IS A RESIDUAL RISK.



IT IS COMPULSORY TO INSTALL THE MACHINE IN A PLACE OF USE HAVING THE CHARACTERISTICS INDICATED ON PAR. "INTENDED USE OF THE MACHINE".

IT IS COMPULSORY TO USE THE MACHINE IN THE RESPECT OF THE LEGISLATION IN FORCE IN THE COUNTRY OF USE CONCERNING INTRODUCTIONS (NOISE).

IT IS COMPULSORY TO KEEP THE MACHINE OUT OF REACH OF UNAUTHORIZED PERSONS.



IT IS STRICTLY PROHIBITED THE USE OF THE MACHINE FOR MISUSE AND DIFFERENT USES FROM THE ONE INTENDED BY THE MANUFACTURER.

IT IS ABSOLUTELY FORBIDDEN TO USE THE MACHINE OUTDOORS.

IT IS STRICTLY PROHIBITED THE USE OF AIR TUBES (EXTENSIONS), FITTINGS AND NOT SUITABLE TOOLS AND/OR NOT CONFORM TO THE STANDARDS IN FORCE.

IT IS ABSOLUTELY FORBIDDEN TO LIFT THE MACHINE WITH CRANE AND/OR FORKLIFT.

IT IS ABSOLUTELY FORBIDDEN TO LEAD AIR JETS AGAINST PERSONS AND/OR ANIMALS.

DO NOT CLIMB ONTO THE MACHINE.



IT IS STRICTLY PROHIBITED TO COMMISSION THE MACHINE IN ENVIRONMENTS WITH POTENTIALLY EXPLOSIVE ATMOSPHERE AND/OR IN THE PRESENCE OF COMBUSTIBLE DUSTS (E.G.: WOOD POWDER, FLOUR, SUGARS AND GRAINS).

IT IS ABSOLUTELY FORBIDDEN TO USE THE MACHINE IN AREAS CONTAINING SOLID OR ABRASIVE PARTICLES.

TRANSPORTATION AND HANDLING

TRANSPORTATION OF THE MACHINE



The machine is delivered to the customer through a “specialized transportation company” or by the manufacturer who, through its own personnel and appropriate means, in compliance with the regulations, shall ensure the operations of packaging, lifting, loading, transport and unloading in relation to the type of transport (land, sea or air).



ALL OPERATIONS OF LIFTING, LOADING, TRANSPORT AND UNLOADING OF THE MACHINE REQUIRE TO BE PERFORMED BY A “SPECIALIZED TRANSPORT COMPANY” OR BY THE MANUFACTURER WITH PERSONNEL AND MEANS SUITABLE FOR USE.

PACKAGING



The machine is fixed to the pallet by straps and covered with a cardboard box. The packaging contains:



- No. 1 compressor
- No. 1 instruction manual

The characteristics of the packaging are indicated in **Tab. 1**.

Mod. CLINIC	1.25	DRY 1.25	3.25	DRY 3.25
Packaging Weight	47 Kg	53 Kg	49 Kg	76 Kg
Pack measurement (machine + bed height)	68 × 57 × 90 cm			

Tab. 1



HANDLING OF PACKAGING WITH A FORKLIFT OR TRANSPALLET



IT IS MANDATORY THAT, IN THE RANGE OF HANDLING OPERATIONS, THERE ARE NO PEOPLE, ANIMALS AND/OR THINGS WHOSE SAFETY CAN BE COMPROMISED ACCIDENTALLY.



THE HANDLING OF THE PACKAGE CAN BE PERFORMED ONLY BY THE USE OF A TRANSPALLET OR A FORKLIFT SUITABLE FOR USE.

IT IS MANDATORY, DURING THE LIFTING OPERATIONS, (UP/DOWN), NOT TO MAKE SUDDEN MANOEUVRES AFFECTING STABILITY OF LOAD.


- 1) Insert the forks (2 - Fig. 1) under the pallet (1), keeping them at the centre of the latter.
- 2) Raise the packaging (3) a few centimetres to make sure the load is actually centred in the forks.
- 3) Carefully move the package to the positioning area expected, therefore lowering the load until the full stability is reached.

UNPACKING

After laying the packaging on the ground do the unpacking as follows:

- 1) With a suitable tool, cut the straps (4) taking care not to be affected by the straps as a result.
- 2) Remove the cardboard box.

BE SURE TO DISPOSE THE PACKAGING MATERIAL ACCORDING TO THE DIFFERENT TYPES OF MATERIAL IN THE ABSOLUTE RESPECT OF THE LEGISLATION IN FORCE IN THE COUNTRY OF USE.

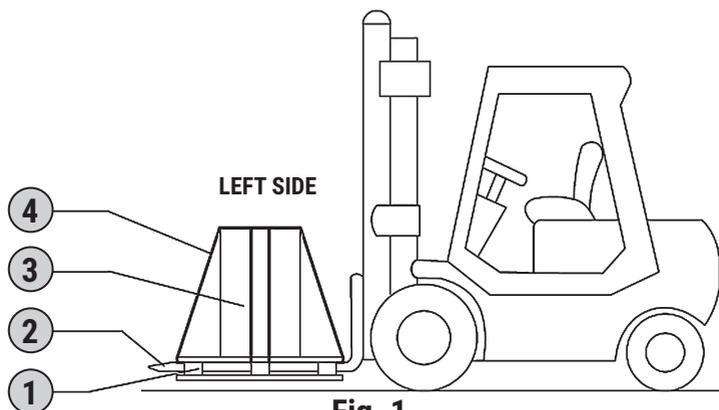



MANUAL HANDLING OF THE MACHINE

! THE MANUAL HANDLING OF THE MACHINE MUST BE DONE BY AT LEAST TWO OPERATORS IN CHARGE, OR USING A SUITABLE MEAN (E.G.: TRANSPALLET), IN ACCORDANCE WITH THE REGULATIONS ON THE “MANUAL HANDLING OF LOADS”, IN ORDER TO AVOID ADVERSE CONDITIONS THAT INVOLVE RISK OF ERGONOMIC INJURIES TO THE BACK.

The machine can be moved manually by two operators in charge, using the handles (1 - Fig. 1) that are attached. Make sure the plug of the power supply cable is disconnected and that the cord is positioned above the tank.

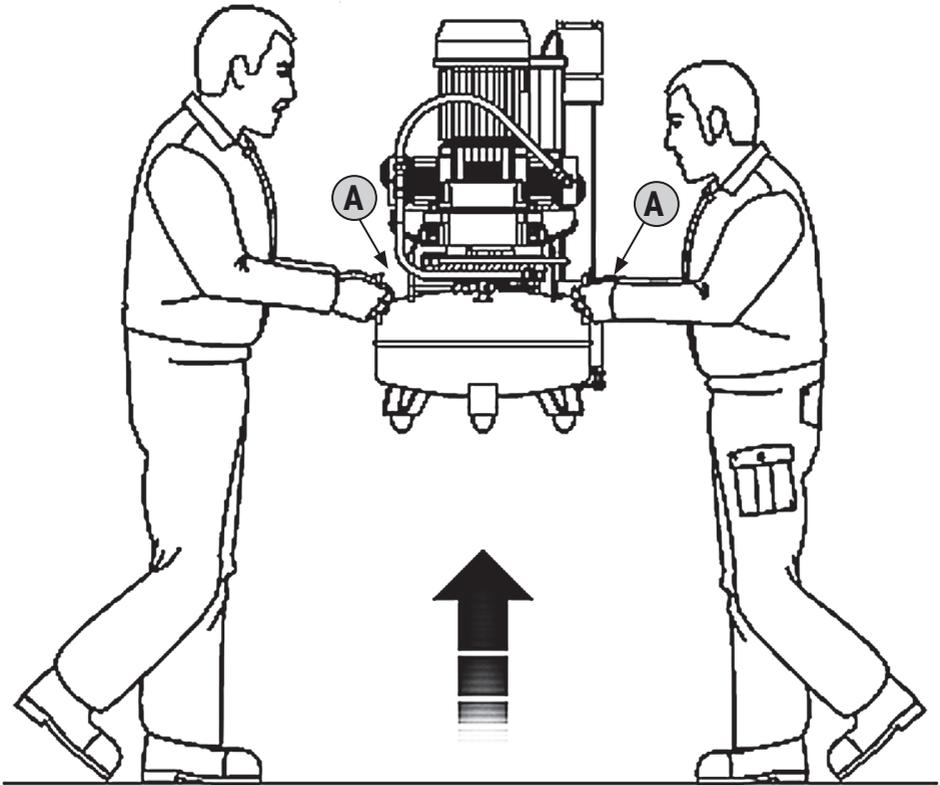


Fig. 1



POSITIONING OF THE MACHINE

 The machine must be placed in a workplace with the characteristics described in par. "INTENDED USE OF THE MACHINE", placed on a flat floor that ensures stability in relation to the overall dimensions and to the weight.



 **IN ORDER TO ENSURE THAT THE AUTHORIZED OPERATORS OCCUPY THE WORK POSITIONS EXPECTED, IT IS MANDATORY TO ENSURE MINIMUM MEASURES OF POSITIONING AS SHOWN IN FIG. 2. IF THE ROOM TEMPERATURE, WHERE THE MACHINE HAS BEEN PLACED, EXCEEDS THE LIMITS SPECIFIED IN PAR. "TECHNICAL SPECIFICATIONS", IT IS COMPULSORY TO INSTALL A SUITABLE ADDITIONAL FAN, AS SHOWN IN FIG. 2.**

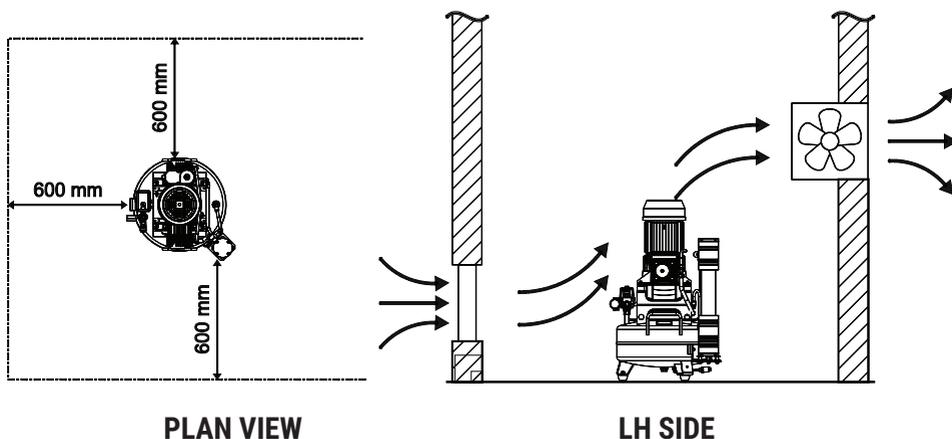


Fig. 2



INSTALLATION

GENERAL WARNINGS

! IT IS COMPULSORY TO COMPLY WITH THE CORRECT PROCEDURES OF ASSEMBLY/DISASSEMBLY AND CONNECTION OF COMPONENTS DESCRIBED IN THIS CHAPTER.

THE MANUAL HANDLING MUST BE IN COMPLIANCE WITH THE REGULATIONS ON THE “MANUAL HANDLING OF LOADS” IN ORDER TO AVOID ADVERSE CONDITIONS THAT INVOLVE RISK OF INJURIES TO THE BACK.

ALL OPERATIONS DESCRIBED BELOW ARE DONE TO PREPARE THE MACHINE READY FOR THE OPERATION.

PNEUMATIC CONNECTION

! THE MANUFACTURER HAS NO LIABILITY FOR INJURY TO PERSONS, ANIMALS AND/OR THINGS, RESULTING FROM FAILURE TO OBSERVE THE WARNINGS DESCRIBED ABOVE.

⊘ IT IS STRICTLY PROHIBITED THE USE OF AIR TUBES (EXTENSIONS), FITTINGS AND TOOLS NOT SUITABLE AND NOT CONFORM TO THE REGULATIONS IN FORCE.

! IT IS COMPULSORY TO USE AIR TUBES (EXTENSIONS), FITTINGS AND TOOLS, ACCORDING TO WHAT IS FORECAST IN THE “INSTRUCTION MANUALS” PROVIDED BY THE MANUFACTURERS.

! The machine must be connected upstream to a pneumatic system for the production sterilized air (by customer) connecting, to the air outlet (**Fig. 1**), the tube for air inlet of the system equipped with a special fitting.

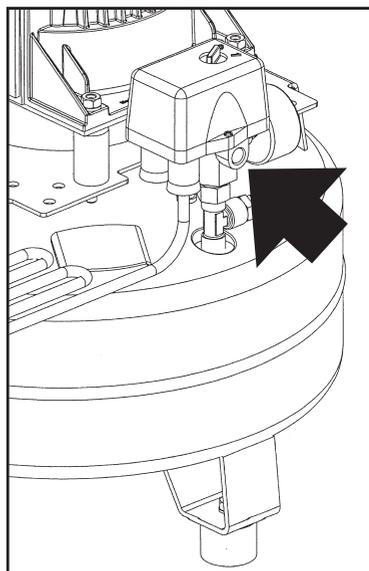


Fig. 1



ELECTRIC CONNECTION

 **THE MANUFACTURER HAS NO LIABILITY FOR FAILURE OR MALFUNCTIONS OF OPERATION TO THE MACHINE CAUSED BY VOLTAGE DROPS EXCEEDING THE TOLERANCES PROVIDED BY THE DISTRIBUTING BODY ($\pm 10\%$ POWER - FREQUENCY $\pm 2\%$).**

 **THE MAINS TO WHICH THE MACHINE IS CONNECTED MUST BE IN ACCORDANCE WITH THE REQUIREMENTS OF THE LAWS OF THE COUNTRY OF USE, MEET THE SPECIFICATIONS CONTAINED IN PAR. "MACHINE DATA" AND BE PROVIDED WITH A SUITABLE GROUNDING SYSTEM.**

IT IS COMPULSORY TO INSTALL UPSTREAM OF THE POWER SUPPLY LINE AN ELECTRIC CUTTING DEVICE WITH A SUITABLE DISCONNECTING PROTECTION COORDINATED WITH THE GROUNDING PLANT.


ANY TYPE OF ELECTRIC MATERIAL USED FOR THE CONNECTION MUST BE SUITABLE FOR USE, MARKED "CE" IF SUBJECT TO LOW VOLTAGE DIRECTIVE 2006/95/EC AND COMPLYING WITH THE REQUIREMENTS OF THE REGULATIONS IN FORCE IN THE COUNTRY OF USE OF THE MACHINE.

IF NECESSARY, IT IS COMPULSORY TO CONNECT THE MACHINE ONLY TO GENERATORS OF ELECTRICITY WITH A POWER GREATER THAN THE POWER SUPPLY INSTALLED, TO SUPPORT THE ABSORPTION CURRENT UPON STARTUP.

FAILURE TO FOLLOW THE WARNINGS ABOVE MAY CAUSE IRREVERSIBLE DAMAGE TO THE ELECTRIC EQUIPMENT OF THE MACHINE AND THE SUBSEQUENT VOID OF THE WARRANTY.



The machine can be connected to the mains by plugging the power cord (11) into the power outlet (by the Customer).

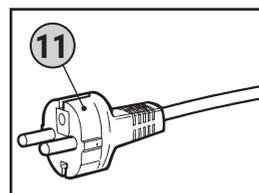



SAFETY

SAFETY DEVICES

The Safety Devices adopted are as follows (for the references on the drawings, please see the pages down):

REF.	DESCRIPTION	FUNCTION
1	Resettable thermal.	It is a thermal protection installed on the electrical system, which stops the electric motor in case of current overload and/or short circuit. Recovery can be done through the appropriate button.
2	Tank pressure gauge.	It is a measuring device installed on the tank of the machine. It shows the air pressure inside the tank (0 ÷ 10 bar).
3	Safety valve.	It is a certified safety valve (set at 10.5 bar), installed in the pneumatic system under the pressure switch. It is used to discharge the overpressure of the system if the pressure switch, for any problems, does not function. The intervention of the safety valve forces the operator to turn off the machine and request the assistance of the maintenance personnel (see par. "INTERVENTION OF THE SAFETY VALVE").
4	Pressure switch.	It is an electro-pneumatic device (set at min. 6 bar, max. 8 bar for Mod. CLINIC / CLINIC DRY 1.25 and at min. 8 bar max. 10 bar for Mod. CLINIC / CLINIC DRY 3.25) fitted onto the pneumatic plant. It is used to control the automatic start of the machine when the pressure drops to 6 bars for Mod. CLINIC / CLINIC DRY 1.25 and to 8 bars for Mod. CLINIC / CLINIC DRY 3.25 and the automatic stop when the pressure reaches 8 bars for Mod. CLINIC / CLINIC DRY 1.25 and 10 bars for Mod. CLINIC / CLINIC DRY 3.25.



IT IS COMPULSORY TO CONSTANTLY CHECK THE GOOD FUNCTIONING AND THE EFFICIENCY OF ALL SAFETY DEVICES IN THIS MACHINE.


IT IS COMPULSORY TO REPLACE ANY SAFETY DEVICES MALFUNCTIONING AND/OR DAMAGED.




⊘ IT IS FORBIDDEN TO MANIPULATE, EXCLUDE, REMOVE AND/OR REPLACE ANY SAFETY DEVICE IN THIS MACHINE.

IT IS ABSOLUTELY FORBIDDEN TO REPLACE ANY SAFETY DEVICE OR COMPONENT WITH NON ORIGINAL SPARE PARTS.


CLINIC DRY 1.25

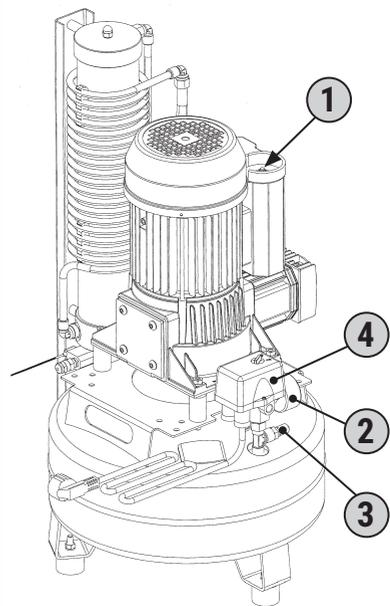


CLINIC 1.25

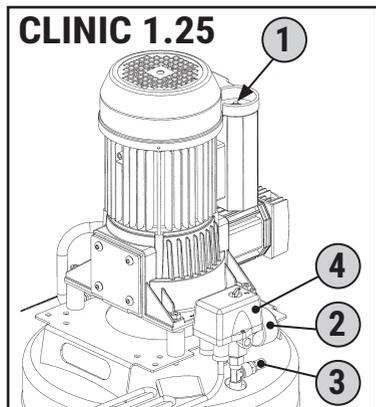

CLINIC DRY 3.25

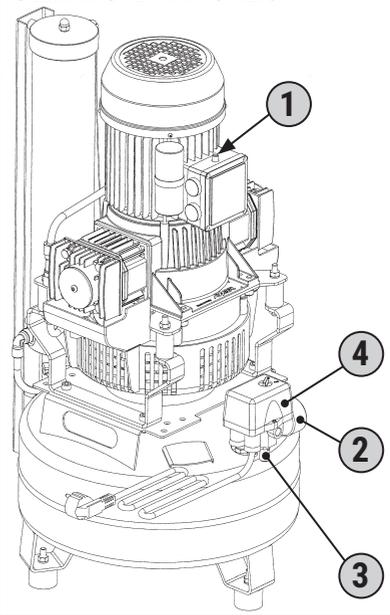

CLINIC 3.25

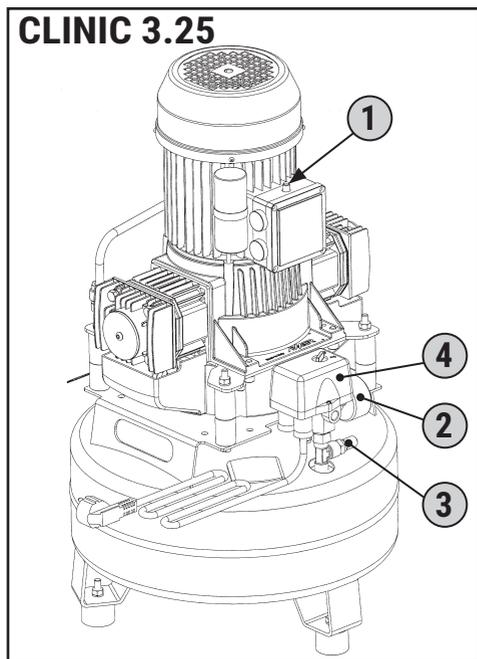



PERSONAL PROTECTIVE EQUIPMENT (PPD)

The use of personal protective equipment (PPD) is mandatory, in accordance with the legislation on safety and health in the workplace regulations of the country of use of the machine.

SEGN	PPE REQUIRED	TYPE OF APPLICATION
	Protection of hands (protective gloves against thermal hazards)	During handling, installation, operation and maintenance.
	Protection of feet (shoes with reinforced toe)	

Tab. 1

 **IT IS COMPULSORY TO USE THE PPD SUGGESTED BY THE MANUFACTURER (SEE Tab. 1).**

RESIDUAL RISKS

The authorized operator must remark that, although the manufacturer has taken every precaution technically possible to make the machine safe, in order not to compromise the functionality and productivity of the machine, there are still two potential residual risks described in **Tab. 2** and **3**.

RESIDUAL RISK N. 1	DANGER OF BURNS IN THE PUMPING GROUP AND ELECTRIC MOTOR
SEVERITY OF THE INJURY	Light, normally reversible injuries.
EXPOSURE	If the operator decides to perform a voluntary improper action, forbidden and not reasonably foreseeable.
PROBABILITY	Low and accidental.
STAGE WHERE THERE IS A RISK	During use and operation.
MEASURES TAKEN	Safety signs. Compulsory use of personal protective equipment (PPD) and/or wait for the cooling of the machine.

Tab. 2



RESIDUAL RISK N. 2	DANGER OF CRUSHING FOOT DUE TO ACCIDENTAL FALL FROM THE MACHINE
SEVERITY OF THE INJURY	Serious injuries usually reversible.
EXPOSURE	If the operator decides to perform a voluntary improper action, forbidden and not reasonably foreseeable.
PROBABILITY	Low.
STAGE WHERE THERE IS A RISK	During handling and placement.
MEASURES TAKEN	Procedures for handling of the packaging and machine (see chap. "TRANSPORTATION AND HANDLING").

Tab. 3

SAFETY SIGNS

The Safety signs used are constituted by an adhesive label, applied externally to the machine (Fig. 1).


 **IT IS ABSOLUTELY FORBIDDEN TO REMOVE AND/OR DAMAGE THE SAFETY SIGNS APPLIED TO THE MACHINE.**

 **IT IS COMPULSORY TO KEEP WELL CLEAR THE SIGNS IN ORDER TO ENSURE A GOOD VISIBILITY.**

IT IS COMPULSORY TO REPLACE THE SAFETY SIGNS DETERIORATED, BY REQUESTING THEM TO THE MANUFACTURER AND/OR AUTHORIZED DEALER.



Fig. 1

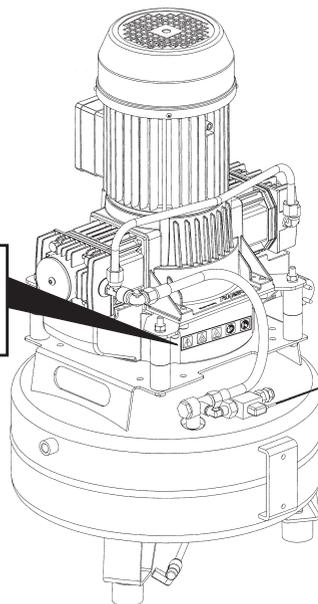



SIGN	MEANING/PRESCRIPTION	SIGN	MEANING/PRESCRIPTION
	DANGER: Electric Current		REQUIREMENT: Read the instruction manual
	DANGER: Automatic start		
	DANGER: Dangerous Temperatures		REQUIREMENT: Turn off the machine before carrying out maintenance

USE AND OPERATION

PLACES OCCUPIED BY AUTHORIZED OPERATORS

The Authorized operators, depending on the type of manual intervention required, must take the places shown in **Fig. 1**.

THE AUTHORIZED OPERATORS ACCORDING TO THE PLACE OCCUPIED, ARE OBLIGED TO PERFORM THE REQUIRED ACTIONS DESCRIBED IN THE RELATED SECTIONS.

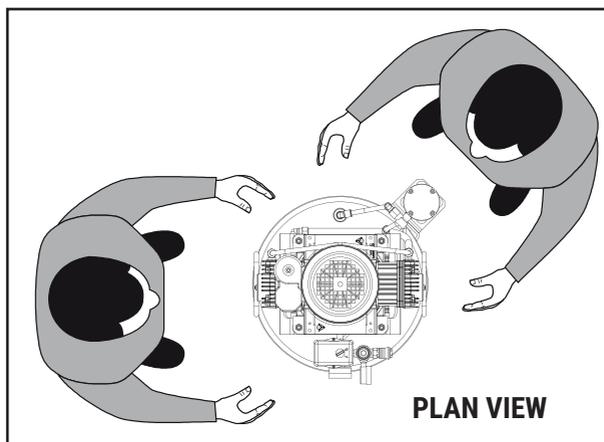


Fig. 1

AUTHORIZED OPERATORS	PLACE	DESCRIPTION OF INTERVENTION
 	ALL	With the consent of the Supervisor, execute on the machine lifting operations, maintenance operations, handling and use of the machine.



PRESSURE SWITCH

In **Fig. 2** is represented the control device to which the authorized operator must refer for the command and control of the machine.


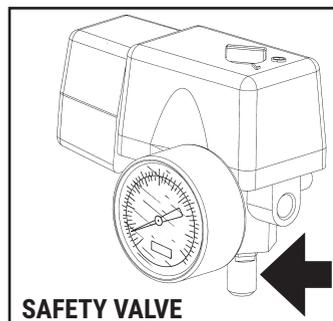



Fig. 2 SAFETY VALVE

REF.	NAME	FUNCTION
1	Fitting.	Blind fitting for accessories.
2	Tank pressure gauge.	It shows the outlet air pressure set through the regulator.
3	Pressure switch.	It is an electro-pneumatic device (set at min. 6 bar, max. 8 bar for Mod. CLINIC / CLINIC DRY 1.25 and at min. 8 bar max. 10 bar for Mod. CLINIC / CLINIC DRY 3.25) fitted onto the pneumatic plant. It is used to control the automatic start of the machine when the pressure drops to 6 bars for Mod. CLINIC / CLINIC DRY 1.25 and to 8 bars for Mod. CLINIC / CLINIC DRY 3.25 and the automatic stop when the pressure reaches 8 bars for Mod. CLINIC / CLINIC DRY 1.25 and 10 bars for Mod. CLINIC / CLINIC DRY 3.25.
4	Interruttore "ON (I)-OFF (O)"	Pos. "ON (I)": Turns on the machine; Pos. "OFF (O)": Turns off the machine.
5	Safety valve.	It is a certified safety valve (set at 10.5 bar), installed in the pneumatic system under the pressure switch. It is used to discharge the overpressure in the system if the pressure switch, for any problems, does not function. The intervention of the safety valve forces the operator to turn off the machine and request the assistance of the maintenance technicians.




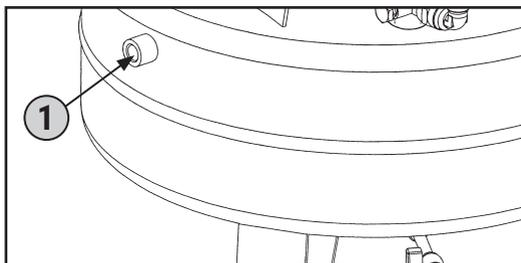


CLINIC 1.25

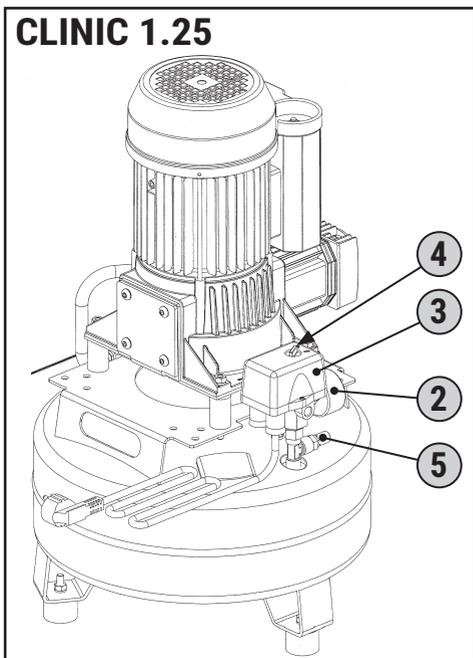


CLINIC DRY 1.25

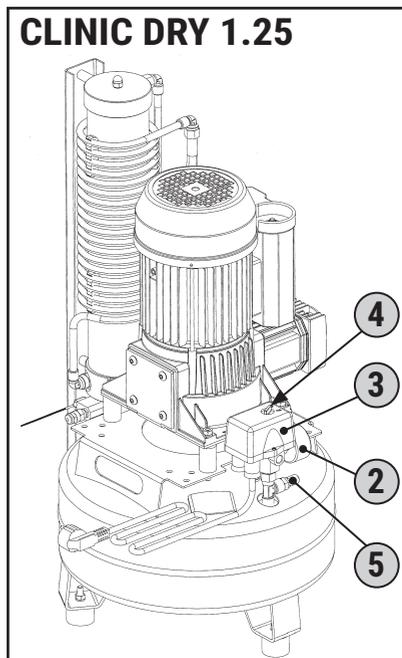

CLINIC 3.25

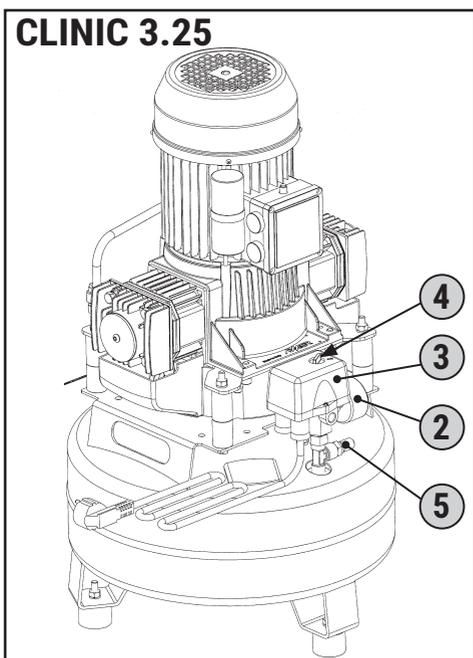

CLINIC DRY 3.25

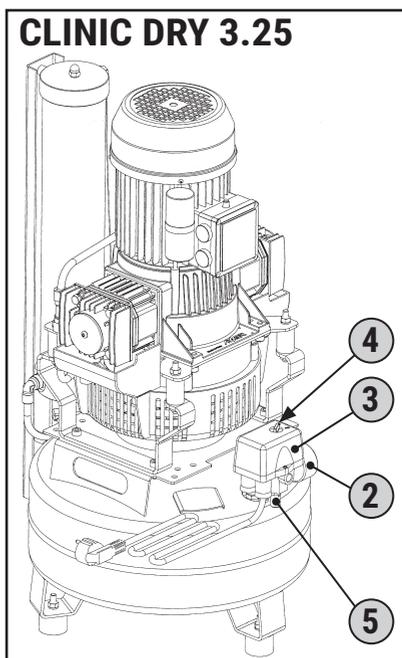



PRIOR TO IGNITION





BEFORE TURNING ON THE MACHINE THE OPERATOR IN CHARGE MUST MAKE THE CHECKS INDICATED IN TAB. 1.



No.	CONTROL
1	Make sure that no unauthorized persons are near the machine.
2	Make sure the machine is properly positioned.
3	Make sure all connections are properly made.
4	Ensure that the switch (14) is in pos. "OFF (0)".
5	Make sure the condensate drain valve is closed (9).
6	Use the Personal protective equipment (PPE) required.
7	Make sure you have read and understood all parts of this "Instruction Manual".
8	Make sure that after last use routine maintenance has been made.

Tab. 1

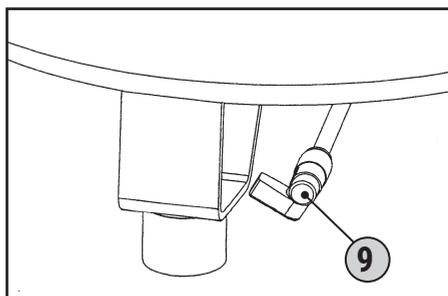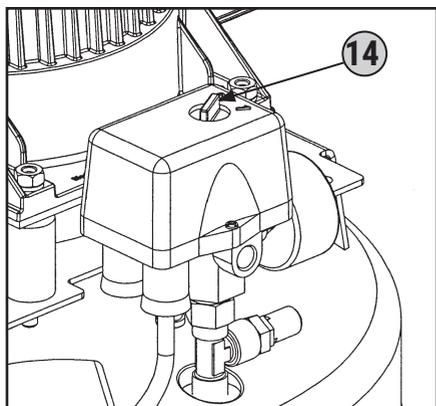

TURNING ON THE MACHINE



THE AUTHORIZED OPERATORS CAN TURN THE MACHINE ON ONLY AFTER THEY HAVE PERFORMED AS A COMPULSORY OPERATION, THE CONTROLS DESCRIBED IN THE CONTROLS TABLE BEFORE USING THE MACHINE IN



QUESTION IT IS COMPULSORY THAT THE AUTHORIZED OPERATORS READ AND THOROUGHLY UNDERSTAND IN ALL PARTS THIS MANUAL. IT IS COMPULSORY TO KEEP THE MACHINE OUT OF REACH OF UNAUTHORIZED PERSONS.



⊘ IT IS FORBIDDEN TO THE AUTHORIZED OPERATORS TO LEAVE THE MACHINE WITHOUT SUPERVISION DURING THE MAINTENANCE WORKS.

THE MANUFACTURER HAS NO LIABILITY FOR INJURIES TO PERSONS, ANIMALS AND/OR THINGS CAUSED BY THE INOBSERVANCE OF THE RULES AND INFORMATION DESCRIBED HEREIN.

Turn on the machine by moving the switch (14) in pos. "ON (I)" (the machine runs up to the max. operating pressure at 8 bar for Mod. CLINIC / CLINIC DRY 1.25 and at 10 bar for Mod. CLINIC / CLINIC DRY 3.25 and then stops automatically).



RESETTABLE THERMAL

👤 If there is an overload and/or a short circuit in the electrical system of the machine, the overload protection intervenes stopping the electric motor.

The reset will occur automatically when the temperature returns within the parameters preset by the manufacturer:

- 1) Set the switch (14) in pos. "OFF (0)".
- 2) Press the button of the resettable thermal (5).
- 3) Before restarting the machine wait a few minutes.

⚠️ IF AFTER PERFORMING THE RESET THE MACHINE WILL NOT RESTART, THE OPERATOR IS REQUIRED TO OBTAIN THE INTERVENTION OF MAINTENANCE TECHNICIANS AND/OR THE AUTHORIZED DEALER.

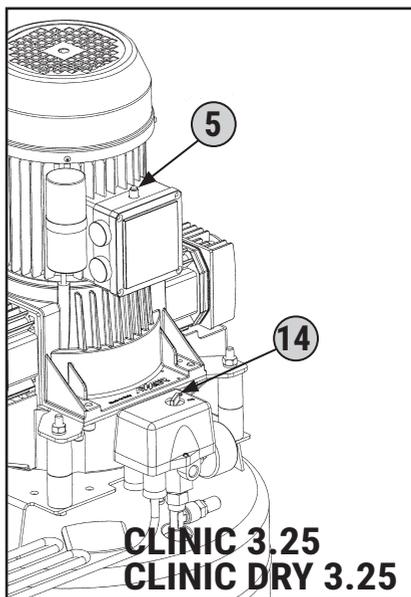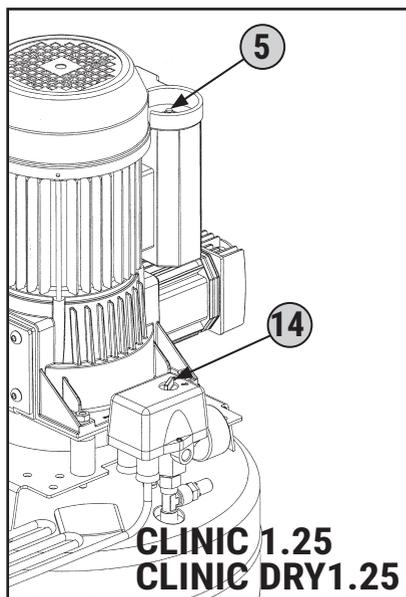



TURNING OFF THE MACHINE



At the end of the work cycle perform the turning off of the machine as follows:



1) Set the switch (14) in pos. "OFF (0)".



2) Unplug the power cord (11) from the socket and place it next to the machine.

3) Only in case of work cycle end perform the condensate discharge from the tank, acting on the condensate drain valve (9).

EMERGENCY STOP






The Emergency Stop of the machine can be done by bringing in pos. "OFF (0)" the switch (14). To avoid situations of imminent danger, the Authorized



Operators are obliged to perform the following operations:

1	RUN IMMEDIATELY THE TURNING OFF OF THE MACHINE AS DESCRIBED IN PAR. "TURNING OFF THE MACHINE".
2	REPORT IMMEDIATELY THE EMERGENCY TO THE SUPERVISOR IN CHARGE.

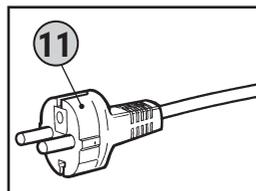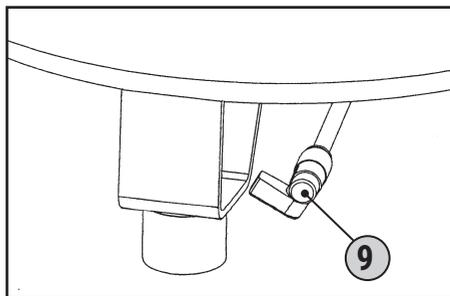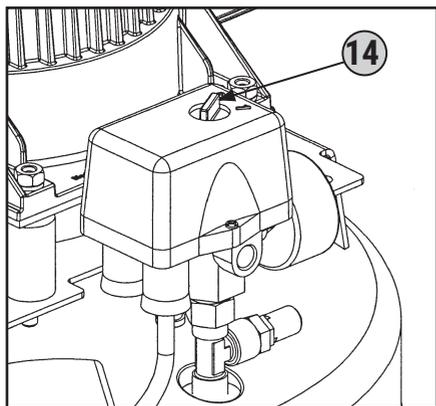



ACTIVATION AFTER AN EMERGENCY STOP


 Only after removing the cause of the emergency and having carefully evaluated that the decision has not caused any damage and/or problems to the machine, with the consent of the Supervisor in charge restore the machine as described in the following paragraph:

- 1) Set the switch (14) in pos. "OFF (0)".
- 2) If necessary, repeat the operations described in par. "TURNING ON THE MACHINE".

INTERVENTION ON THE SAFETY VALVE

 In the event of a malfunction of the pressure switch and the pressure inside the tank exceeds 10,5 bar, the safety valve operates mechanically by opening for venting outside the compressed air and return the pressure to the safety value.

 **UPON THE INTERVENTION OF THE SAFETY VALVE IT IS COMPULSORY TO TURN OFF THE MACHINE, NOTIFY THE SUPERVISOR AND REQUEST ASSISTANCE OF MAINTENANCE TECHNICIANS OR AUTHORIZED DEALER TO MAKE THE CONTROL OF PRESSURE SWITCH.**

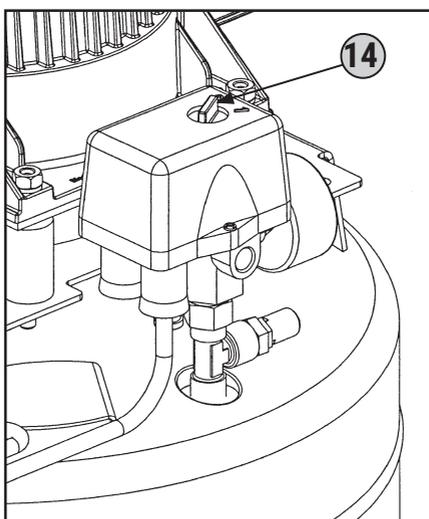





TROUBLESHOOTING



⚠ FAILURE TO FOLLOW THE PROCEDURES CONTAINED IN THIS PARAGRAPH MAY CAUSE A MALFUNCTION OF THE MACHINE AND/OR UNDERMINE THE SAFETY OF THE SAME.

PROBLEM	CAUSE	SOLUTIONS
The machine does not turn or stops and does not restart.	No electricity.	<ul style="list-style-type: none"> • Check that the switch (14) is in pos. "ON (I)". • Check that the power cord plug (11) is working properly and inserted into the socket. • Check that any extension cords used and the current outlet are operational. • Check that the switch of the main power supply is functioning and in pos. "ON (I)".
The machine does not turn or stops and does not restart.	Intervention of the re-settable thermal due to current overload and/or short circuit.	<ul style="list-style-type: none"> • Follow the procedure described in par. "RESETTABLE THERMAL".
The machine often starts again without the pneumatic plant is operating.	Air leaks from the pipe or its pneumatic plant.	<ul style="list-style-type: none"> • Check that the pneumatic system is not damaged. • Check the integrity of the pipe connections and air system.

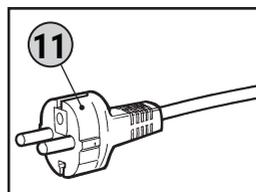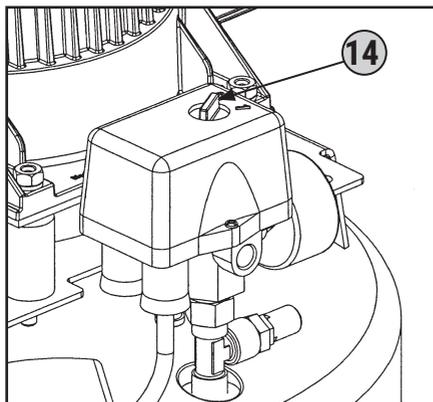



PROBLEM	CAUSE	SOLUTIONS
Intervention of the safety valve.	Faulty pressure switch.	See the maintenance technicians or contact an Authorized Reseller.   
Air leak from the valve of the pressure switch with machine stopped.	Check valve is dirty or worn.	
The machine vibrates and/or makes a lot of noise.	Mechanical failure.	
Frequent starts and low efficiency.	Dirty air filters.	

MAINTENANCE

 This is the set of activities designed to maintain the conditions of use and operation of the machine, through various types of intervention (adjustments, visual inspection, recovery levels, etc.) performed by the maintenance technician or by the manufacturer's technician at the frequency forecast.



 **THE AUTHORIZED OPERATORS SHOULD PERFORM ONLY THE MAINTENANCE WORKS REQUIRED IN ACCORDANCE WITH THEIR SPECIFIC PROFESSIONAL COMPETENCE AND UPON CONSENT OF THE SUPERVISOR IN CHARGE.**

 **IT IS FORBIDDEN TO AUTHORIZED OPERATORS TO LEAVE THE MACHINE WITHOUT SUPERVISION DURING THE MAINTENANCE WORKS. THEY SHOULD ALSO PROTECT THE PLACE OF WORK WITH A WHITE/RED CHAIN AND EXPOSE ADEQUATE NOTICE OF DANGER AND PROHIBITION FOR WORKS IN PROGRESS.**

	POINT OF INTERVENTION	TANK
	TYPE OF INTERVENTION	CONDENSATE DRAIN



EVERY WEEK




At the end of the workday, run the condensate drain of the tank, following the steps below:



- 1) Place under the condensate drain valve, a collection container fit for use.
- 2) Slowly open the valve of condensate drain (12) and wait for a complete exit of compressed air and condensate.
- 3) Close the drain valve.
- 4) Indicate the action on the maintenance card.



YOU MUST DISPOSE OF THE CONDENSATION LIQUID IN COMPLIANCE WITH THE REGULATIONS IN FORCE IN THE COUNTRY OF USE.

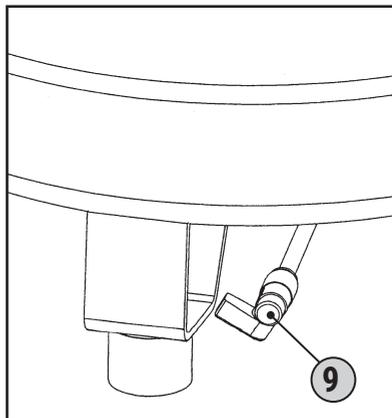



EVERY MONTH



Proceed as follows:

1) Unscrew the screw (1).



2) Remove the cover (2).

3) Remove the filter (3) and clean it with compressed air.




REPLACE THE FILTER IN CASE OF EVIDENT WEAR.

4) After placing back the filter, replace the cover properly.

5) Indicate the action on the maintenance card.



YOU MUST DISPOSE OF THE FILTERS REPLACED IN COMPLIANCE WITH THE REGULATIONS IN FORCE IN THE COUNTRY OF USE.

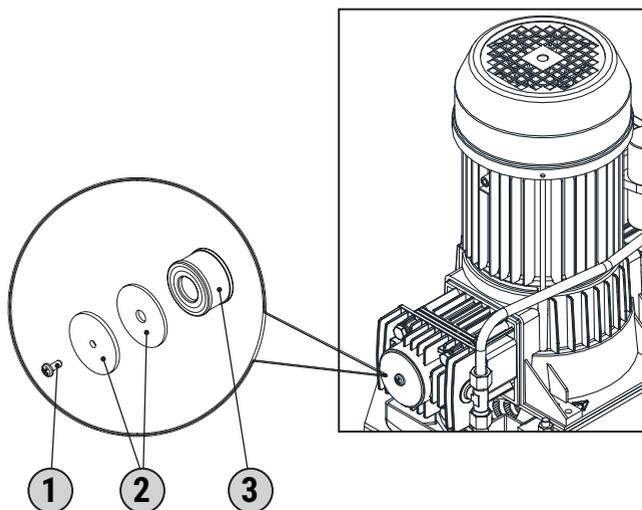


	POINT OF INTERVENTION	COALESCING FILTER
	TYPE OF INTERVENTION	REPLACE



DRYER FILTER ESC.F1

MAINTENANCE EVERY 1000 HOURS OR EVERY 12 MONTHS

- Replacement of PELLETS – REGENERATING KIT (COD. 60610002)
- Unscrew the dryer from the beam (17).
- Unscrew the nut (14) keeping the closing cap (2) locked.
- Place a box under the dryer to hold the pellets to be replaced.
- Remove the upper head (13) and extract the spiral (21), the outer tube (12) and the one internal (11).
- Empty the dryer completely from the exhausted pellets, collecting plates (9) and the spring (16).
- Unscrew the stay rod from the closing cap (2) and extract it from the lower head (5).
- Remove the closing plate (3), with the whole valve unit (24), from the lower head (5).
- Blow inside the pipes (11-12) and the heads (5-13) with compressed air.
- Replace all filters (6) and the O-Rings (1-4-7-8).
- Insert the closing cap (2) with the whole valve unit (24), in the lower head (5) with the closing plate (3) being careful to the position of the O-Rings (7-8).
- Reposition the tubes (11-12) in the lower head (5), insert the first plate (9) and fill the INTERNAL tube (11) with new pellets, insert the second plate (9) and the spring (16).
- Insert the spiral (21) with the spacers (22) engaging it to the fitting (23).
- Reposition the upper head (13) being careful to the position of the threaded hole for the fixing it to the beam.
- Replace the copper washer (15) and tightening the nut (14) torque 25 Nm.
- Fasten everything to the spar (17) with screws (18), washers (19) and spacers (20).
- Reconnect the system.
- Verify proper operation by doing attention to the pneumatic seal on the heads (5-13) on the external tube (12) and on the spiral couplings (21).

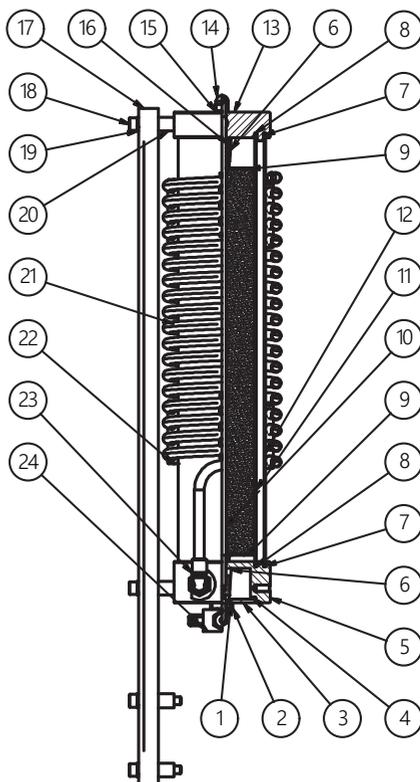




DRYER FILTER ESC.F3

MAINTENANCE EVERY 1000 HOURS OR EVERY 12 MONTHS

- Replacement of PELLETS – REGENERATING KIT (COD. 84213)
- Unscrew the dryer from the beam (17).
- Unscrew the nut (14) keeping the closing cap (2) locked.
- Place a box under the dryer to hold the pellets to be replaced.
- Remove the upper head (13) and extract the outer tube (12) and then the one internal (11).
- Empty the dryer completely from the exhausted pellets, collecting plates (9) and the spring (16).
- Unscrew the stay rod from the closing cap (2) and extract it from the lower head (5).
- Remove the closing plate (3) from the lower head (5).
- Blow inside the pipes (11-12) and the heads (5-13) with compressed air.
- Replace all filters (6) and the O-Rings (1-4-7-8).
- Insert the closing cap (2) in the lower head (5) with the closing plate (3) being careful to the position of the O-Rings (7-8).
- Reposition the tubes (11-12) in the lower head (5), insert the first plate (9) and fill the INTERNAL tube (11) with new pellets, insert the second plate (9) and the spring (16).
- Reposition the upper head (13) being careful to the position of the threaded hole for the fixing it to the beam.
- Replace the copper washer (15) and tightening the nut (14) torque 25 Nm.
- Fasten everything to the spar (17) with screws (18), washers (19) and spacers (20).
- Reconnect the system.
- Verify proper operation by doing attention to the pneumatic seal on the heads (5-13) on the external tube (12).

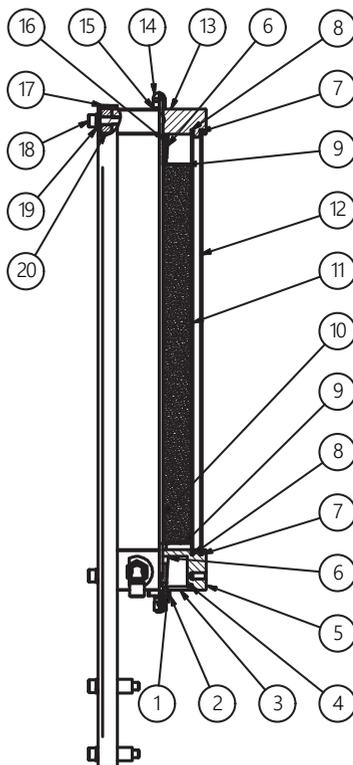



EXTRAORDINARY MAINTENANCE



This is the set of activities designed to maintain the conditions of use and operation of the machine, through various types of intervention (adjustments, changes, etc.) performed by qualified technicians in the event of failure or wear.



FOR ANY INTERVENTION OF EXTRAORDINARY MAINTENANCE IT IS COMPULSORY TO REQUIRE THE TECHNICAL ASSISTANCE TO THE MANUFACTURER OR AUTHORIZED DEALER.

DECOMMISSIONING

STORAGE



In the event that the machine should be stored for some time, you must place it in a safe environment, with a proper degree of temperature, humidity and protected from dust.



DEMOLITION



Upon demolition of the machine, strictly follow the requirements in force in the country of demolition.



Proceed to the differentiation of the parts that make up the machine according to the different types of construction materials (plastics, copper, iron, etc. ...).



These products must be considered pollutants and hazardous charging licensed contractors and disposed of by appointing specialized companies



for different types of products.



ALL PARTS OF THE MACHINE MUST BE IDENTIFIED ACCORDING TO THE DEFINITIONS OF "CER CODES" (EUROPEAN WASTE CATALOGUE) AND DISPOSED OF BY INSTRUCTING AUTHORIZED AND SPECIALIZED COMPANIES, IN STRICT COMPLIANCE WITH THE REGULATIONS IN FORCE IN THE COUNTRY OF DEMOLITION OF THE MACHINE.



WASTES OF ELECTRICAL AND ELECTRONIC EQUIPMENT (RAEE) MUST BE DISPOSED OF IN STRICT COMPLIANCE WITH THE REGULATIONS IN FORCE IN THE COUNTRY FOR THE DEMOLITION OF THE MACHINE.



SPARE PARTS



GENUINE PARTS FOR ANY SUBSTITUTIONS MUST BE REQUIRED ONLY TO THE MANUFACTURER.



IT IS STRICTLY PROHIBITED THE USE OF NON ORIGINAL SPARE PARTS.



MAINTENANCE SCHEDULE CLINIC 1.25

Year	Details		Replacements			Hours
1			/	/	/	500
2	Date	Signature	F2	/	G1	1000
3			/	/	/	1500
4	Date	Signature	F2	/	G1	2000
5			/	/	/	2500
6	Date	Signature	F2	/	G1	3000
7			/	/	/	3500
8	Date	Signature	F2	/	G1	4000
9			/	/	/	4500
10	Date	Signature	F2	/	G1	5000

Description	Q.ty/Code
F2 Air intake filters spare	1×279
G1 Seal gasket kit	1×84301



MAINTENANCE SCHEDULE CLINIC DRY 1.25

Year	Details		Replacements			Hours
1			/	/	/	500
2	Date	Signature	F2	F3	G1	1000
3			/	/	/	1500
4	Date	Signature	F2	F3	G1	2000
5			/	/	/	2500
6	Date	Signature	F2	F3	G1	3000
7			/	/	/	3500
8	Date	Signature	F2	F3	G1	4000
9			/	/	/	4500
10	Date	Signature	F2	F3	G1	5000

Description	Q.ty/Code
F2 Air intake filters spare	1×279
F3 Regenerating dryer kit	1×84213
G1 Seal gasket kit	1×84301



MAINTENANCE SCHEDULE CLINIC 3.25

Year	Details		Replacements			Hours
1			/	/	/	500
2	Date	Signature	F2	/	/	1000
3			/	/	/	1500
4	Date	Signature	F2	/	G1	2000
5			/	/	/	2500
6	Date	Signature	F2	/	/	3000
7			/	/	/	3500
8	Date	Signature	F2	/	G1+G2	4000
9			/	/	/	4500
10	Date	Signature	F2	/	/	5000

Description	Q.ty/Code
F2 Air intake filters spare	2×279
G1 Seal gasket kit	1×84303
G2 Piston Kit	1×83214



MAINTENANCE SCHEDULE CLINIC DRY 3.25

Year	Details		Replacements			Hours
1			/	/	/	500
2	Date	Signature	F2	F3	/	1000
3			/	/	/	1500
4	Date	Signature	F2	F3	G1	2000
5			/	/	/	2500
6	Date	Signature	F2	F3	/	3000
7			/	/	/	3500
8	Date	Signature	F2	F3	G1+G2	4000
9			/	/	/	4500
10	Date	Signature	F2	F3	/	5000

Description	Q.ty/Code
F2 Air intake filters spare	2x279
F3 Regenerating dryer kit	1x84213
G1 Seal gasket kit	1x84303
G2 Piston Kit	1x83214



SINGLE TABLE FOR PACKAGING DISPOSAL D.Lgs. 116/2020 (ITA) - UE 2018/852

<i>D.Lgs. 116/2020 (ITA) - UE 2018/852</i>									
									
IT	Pallet	Scatola (cartone ondulato)	Scatola (cartone pressato)	Foglio	Reggia	Reggia	Busta / Film estensibile	Protezione	Vite / Staffa / Graffa
	LEGNO	CARTA	CARTA	CARTA	PLASTICA	PLASTICA	PLASTICA	POLISTIROLO	FERRO
EN	Pallet	Box (Corrugated Cardboard)	Box (Pressed Cardboard)	Sheet	Strap	Strap	Bag / Stretch Film	Protection	Screw / Bracket / Staple
	WOOD	PAPER	PAPER	PAPER	PLASTIC	PLASTIC	PLASTIC	POLYSTYRENE	IRON
DE	Palette	Karton (Wellpappe)	Karton (Presspappe)	Papier	Band	Band	Beutel/Stretchfolie	Schutz	Schraube/Halterung/Klemme
	HOLZ	PAPIER	PAPIER	PAPIER	KUNSTSTOFF	KUNSTSTOFF	KUNSTSTOFF	STYROPOR	EISEN
FR	Palette	Boîte (carton ondulé)	Boîte (carton comprimé)	Feuille	Feuillard	Feuillard	Enveloppe/Film étirable	Protection	Vis/Attache/Agrafe
	BOIS	PAPIER	PAPIER	PAPIER	PLASTIQUE	PLASTIQUE	PLASTIQUE	POLYSTYRÈNE	FER
ES	Palé	Caja (cartón ondulado)	Caja (cartón prensado)	Hoja	Fleje	Fleje	Sobre / Película extensible	Protección	Tornillo / Estribo / Grapa
	MADERA	PAPEL	PAPEL	PAPEL	PLÁSTICO	PLÁSTICO	PLÁSTICO	POLIESTIRENO	HIERRO
FI	Kuormalava	Laatikko (aaltopahvia)	Laatikko (puristettua pahvia)	Arkki	Vanne	Vanne	Pussi / venytyskalvo	Suojus	Ruuvi / kannatin / kiinnitin
	PUU	PAPERI	PAPERI	PAPERI	MUOVI	MUOVI	MUOVI	POLYSTYREENI	RAUTA
PL	Paleta	Pudełko (tektura falista)	Pudełko (tektura prasowana)	Arkusz	Opaska	Opaska	Torebka/folia stretch	Zabezpieczenie	Śruba/uchwyty/s pinacz
	DREWNO	PAPIER	PAPIER	PAPIER	TWORZYWO SZTUCZNE	TWORZYWO SZTUCZNE	TWORZYWO SZTUCZNE	POLISTYREN	METAL
SV	Lastpall	Låda (wellpapp)	Låda (kartong)	Blad	Tyglina	Tyglina	Kuvert / sträckfilm	Skydd	Skruv / klämma / klammer
	TRÄ	PAPPER	PAPPER	PAPPER	PLAST	PLAST	PLAST	POLYSTIREN	JÄRN



Via delle Tezze, 20/22
36070 TRISSINO (VI)
Italy

phone +39 0445 96 20 00
info@gentilinair.com

www.gentilinair.com



MAN1325001 - 09/2023